




## Implant Spacing Guide Technique Guide

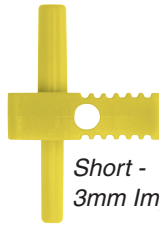

Short -  
3mm Implants

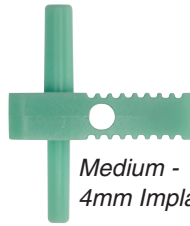


Medium -  
4mm Implants

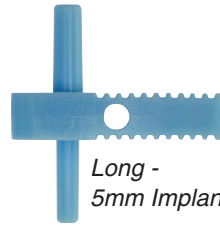


Long -  
5mm Implants


1. Select color coded Guide based on Implant size. Position Guide against tooth adjacent to Implant location.

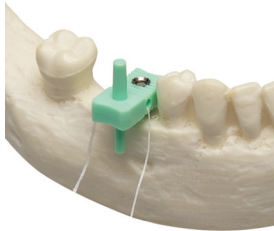


2. Use Perio Probe to locate crest of ridge, then drill pilot hole with a 2.0mm drill.

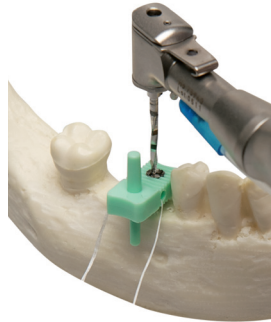


3. Place Salvin Guide Pin into osteotomy to verify spacing and parallelism.

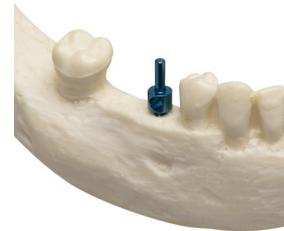

4. Place extension pin into osteotomy and align Guide over ridge.

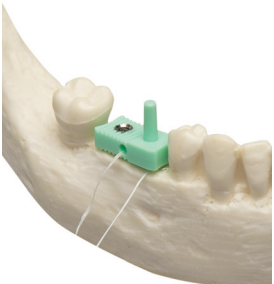

5. Drill second pilot hole through metal ring.

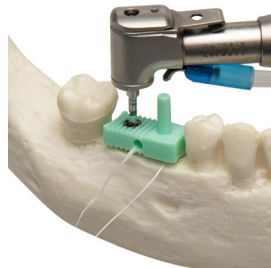

6. Place Salvin Guide Pins into osteotomies to verify spacing and parallelism.

