


No-Clog Surgical Aspirator Technique Guide

The No Clog aspirator features a built in Stylet.

Sliding the button forward causes the stylet to quickly eject particles of bone or tissue that may be clogging the tip.

When the button is returned to its original position, suction again resumes.

This aspirator will fit high volume cut-off valves or surgical tubing.

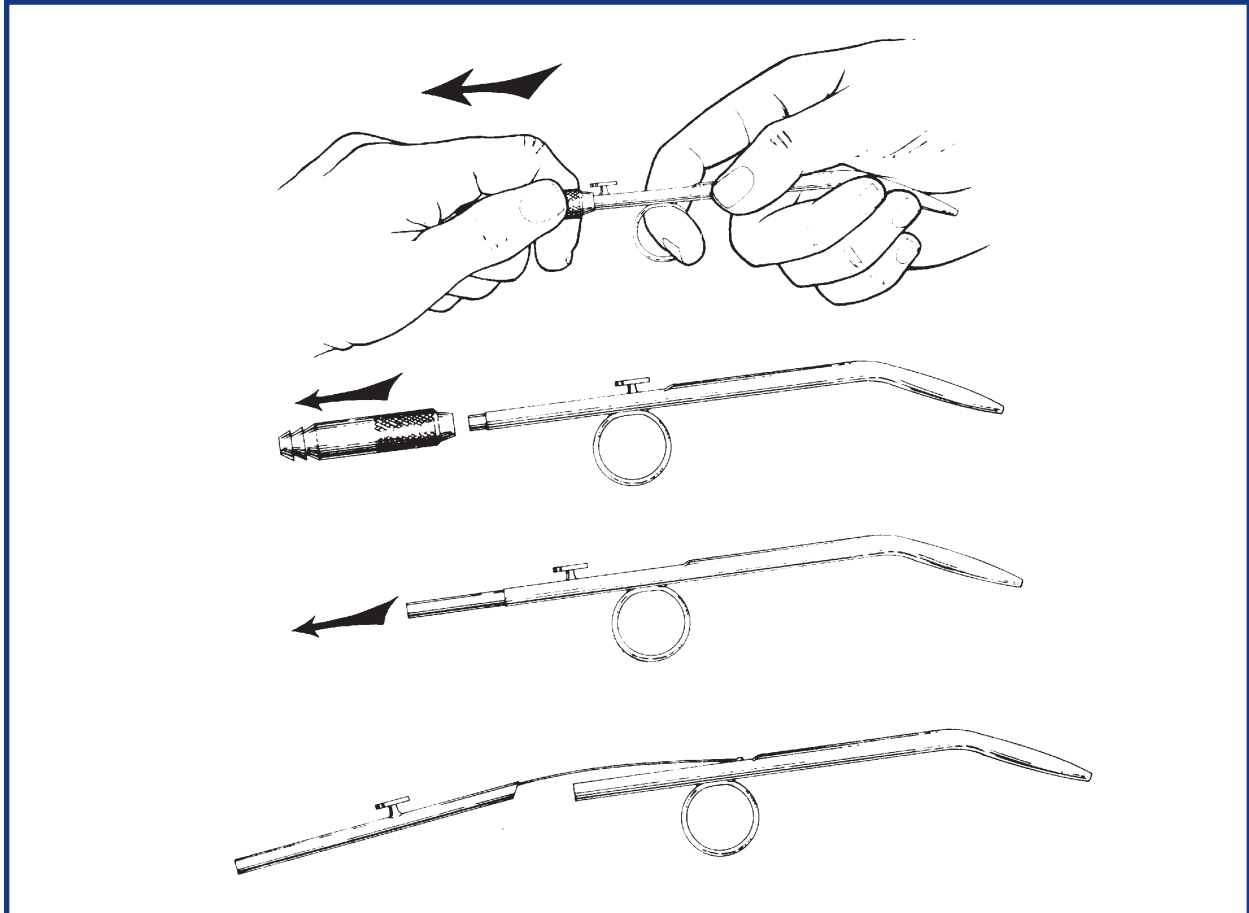

CLEANING AND STERILIZATION

The No-Clog Aspirator is supplied non-sterile and must be cleaned and sterilized before initial use, and re-cleaned and re-sterilized after each use.

Disassemble the No-Clog Aspirator as shown above for cleaning and sterilization.

Please Call Us For:
Aspirator Tip Cleaning Brushes
Autoclavable Surgical Tubing